


Step 6.

Follow instructions of Voice Dialer to setup contacts.

Step 7. (Switch Control for Limited Mobility Clients).

Connect the optional Air or Pillow Switch to the XLC3.4+ Handset.

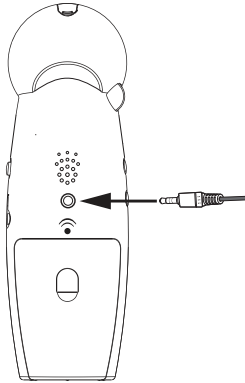

Step 8.

Use the switch control to activate the HANDset. If using the switch control, you must use the included HEADset to operate the XLC3.4+ Handsfree.

- **To answer calls** - Use the switch control to turn on the phone.
- **To end calls** - Use the switch control to turn off the phone.
- **To make calls** - Use the switch control to turn on the phone, and using the headset, activate the voice dialer to dial one of your contacts stored in the voice dialer module.
- **To end calls** - Use the switch control to turn off the phone.

Clarity XLC3.4+HF Quick Setup Guide



**FULLY Charge Batteries in Handset
Before Using (24 Hours Recommended)**

SETUP

Step 1.

Connect the Voice Dialer Power Supply from an Electrical Outlet to the the Voice Dialer Power jack.

Step 2.


Connect the Voice Dialer Telephone Line Long cord from your home Telephone Jack to the Voice Dialer Line jack.

Step 3.

Connect the XLC3.4+ Power Supply from an Electrical Outlet to the XLC3.4+ Base.

Step 4.

Connect the XLC3.4+ Telephone Line Long cord from the XLC3.4+ Base to the Voice Dialer Phone jack.

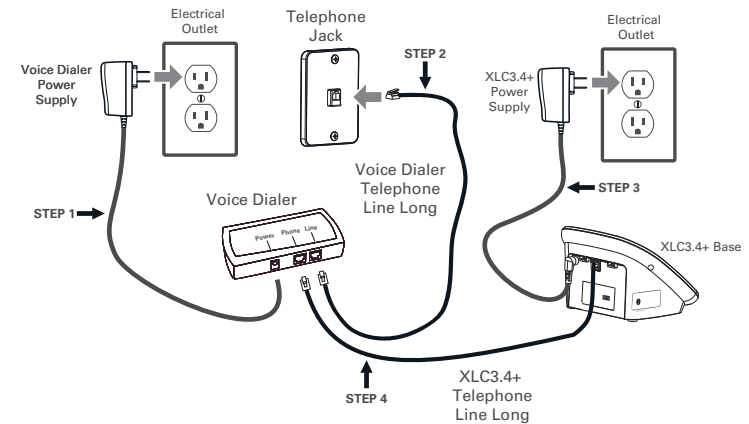

Step 5.

Connect the Teltex HS Smart Headset to XLC3.4+ Handset.



What's in the Box:

- (A) XLC3.4+HF Amplified Cordless Phone
- (B) XLC3.4+HF Power Supply
- (C) XLC3.4+HF Telephone Line Long
- (D) Voice Dialer
- (E) Voice Dialer Power Supply
- (F) Voice Dialer Telephone Line Long
- (G) Pillow Switch (Optional)
- (H) Air Switch (Optional)
- (I) Teltex HS Smart Headset

