



## MFK Touch Function

**Key:** The touch sensor is used to answer & end phone calls. It pauses & plays music. Tap twice to activate the voice control for the second device.

## Volume & Scroll Touch



**Keys:** These keys are used to turn the volume up or down, scroll through the menu or skip forward & back when listening to music.

**MFB Base Multi-Function Button:** This button is used to initiate & answer calls, turn the Quattro 4L on & off, pair the Quattro 4L to Bluetooth devices (e.g. mobile phone), switch between calls and initiate Siri or S-Voice for voice activation.



**8-wire Neckloop:** This cord hangs the Quattro 4L around your neck like a necklace. This is an 8-wire loop that connects to hearing aids or cochlear implants with t-coils, delivering sound directly through the hearing aids or cochlear implants when they are in t-coil mode.


**Earbud Jack:** This jack accepts any 3.5mm audio only headsets. (We suggest using our patented SmartSound Earbuds.)

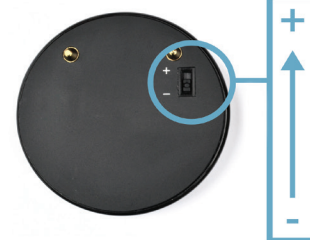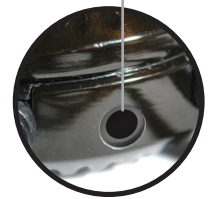

**Battery Engage Switches:** There is a battery engage switch on the Quattro 4L unit. This switch **MUST** be in the **+** position on the Quattro 4L for the unit to function. By switching to the **-** position, the battery is disengaged from the circuit. This switch does **NOT** need to be put in the **-** position; its purpose was to protect the integrity of the battery during storage.

## USING THE QUATTRO 4L

### 1. CHARGING THE QUATTRO 4L

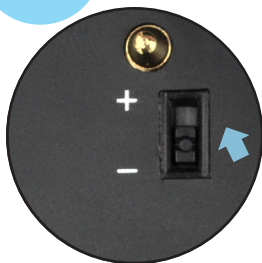

**STEP 1:** Use the tip of a pen to move the battery engagement +/- switch into the + position on the Quattro 4L.

**Do NOT turn it off!**



**STEP 2:** Plug charger into Quattro 4L.



**STEP 3:** Plug charger into outlet to fully charge the Quattro 4L.



**STEP 4:** Red light is on QT4L.



**STEP 5:** Quattro 4L is charged when the red light turns blue.

**TIP:** Charge the Quattro 4L every night for maximum performance. The battery icon will identify when battery power is running low on the main base.

## 2.

### PAIRING THE QUATTRO 4L WITH A BLUETOOTH DEVICE

- Make sure **other connected Bluetooth devices** are **turned off** when syncing a new Bluetooth device.



**STEP 1:** Ensure Quattro 4L is turned off, then press and hold MFB for approximately six seconds.



**STEP 2:** After around six seconds, the Quattro 4L announces "Pairing".



**STEP 3:** Turn on the Bluetooth device and put it in Bluetooth pairing mode. Often this requires device menu selections such as Settings > Bluetooth > ON. Choose Quattro 4L.



**STEP 4:** When the Quattro 4L announces "Paired" then "Device one connected", you are ready to use your Quattro 4L.

## 3.

### TAKE A CALL



**STEP 1:** To answer the call: Firmly press multi-function key once.



**STEP 2:** To reject the call: Press and hold MFB.



**STEP 3:** Adjust the volume: Tap < (volume -) or > (volume +).



**STEP 4:** To end call, press and hold multi-function key. The Quattro 4L announces "call terminated."

**NOTE:** You can place a call directly from your phone with sound delivered through your Quattro 4L if "Headset" or "Quattro 4L" is selected on the phone.