
Quick Start Guide

instaLINK™ Wearable Smart Phone Alert
Model iL-100


Feature Identifications: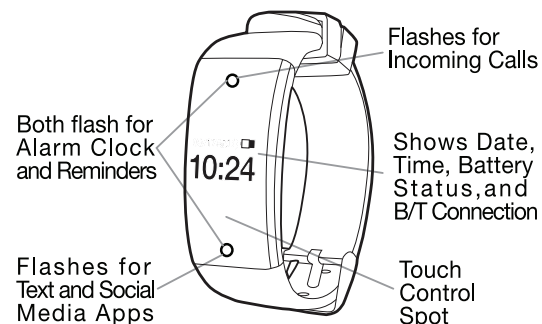

Charging instaLINK:

Charge instaLINK for 2 hours before first use as shown below.

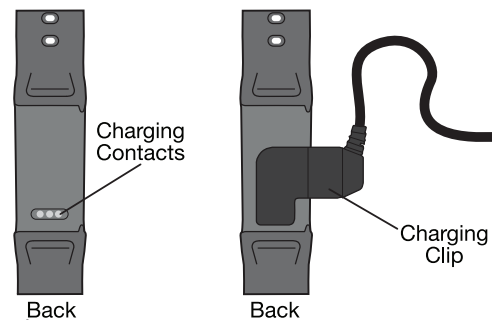Download App:

1. Go to the App Store or Google Play and search for the “SERENE LINK” App by Serene Innovations Inc.
2. Download and install the free App on your iOS or Android device.

Pairing instaLINK to your Smartphone:

1. Turn the instaLINK “ON” by placing finger on touch control spot until the word “Welcome” appears. (fig. A)
2. Make sure your smartphone’s Bluetooth™ function is turned “ON”.

Pairing instaLINK to your Smartphone (CONT.):

3. Tap and open the “SERENE LINK” App. The App will start searching for the nearby instaLINK to complete the pairing process. Press “PAIR” when it appears on screen. (Note: Pairing is done via the App only)
4. When pairing is complete, a small Bluetooth icon will appear on the instaLINK’s display.

Setting Up Alert Notifications:

Follow the App to:

1. Set Date and Time (12 or 24 Hour Clock)
2. Set Alarm Times (Up to 8 Alarms)
3. Set Medicine Reminder Times (Up to 8 Reminders)
4. Select Notifications for:
 - Incoming Calls
 - Text/SMS
 - Social Media Apps: Facebook, Twitter, Skype, Whatsapp, etc.

Alert for Incoming Calls:

1. instaLINK will Vibrate up to 5 Times and then stop. After vibration has stopped, nothing will be shown on the display.

Reviewing Missed Calls:

1. When nothing is on display, tap the Touch Control spot once to show display, then tap touch control spot again.
2. A handset icon will appear on the display.
3. Place finger on Touch Control Spot for 2 seconds again and missed call will appear.
4. Tap the Touch Control Spot once to cycle through 5 most recent missed calls.

Clearing Missed Calls:

1. While reviewing missed calls, place finger on Touch Control Spot for 2 seconds.
2. The message “DELETE ALL” will appear on the display.
3. Place finger on Touch Control Spot for 2 seconds again and all calls will be cleared.
4. After calls are cleared, there will be nothing on the display.

Alert for Text or Social Media Apps:

1. instaLINK will Vibrate 3 Times and then stop. After vibration has stopped, nothing will be shown on the display. Check missed texts and social media App notifications on phone.

Alert for Alarm Clock/Medicine Reminders:

1. InstaLINK will Vibrate up to 5 Times and then stop. After vibration has stopped, nothing will be shown on the display.

Note: If the InstaLINK is out of range for too long (>1hr.), some Android phones may require relaunching of App and then exit.

Bluetooth® is a trademark of Bluetooth SIG, Inc.
APP Store® is a trademark of Apple Inc
Google Play™ is a trademarks of Google Inc.

Need Help? Call Toll-free Help Line:
866-376-9271 (8AM - 5PM PST, M-F)

14731 Carmenita Road, Norwalk, CA 90650
www.sereneinnovations.com

Finished size 3” wide x 4.5” high
Flat size 15” wide x 4.5” high

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