CE30™ AMPLIFIED NECKLOOP

Important Safety Instructions

When using your telephone equipment, basic safety precautions should always be followed to reduce the risk of fire, electric shock and injury to persons, including the following:

- · Read and understand all instructions.
- · Follow all warnings and instructions marked on the product.
- Unplug this product from the wall outlet before cleaning. Do not use liquid cleaners or aerosol cleaners. Use a damp cloth for cleaning.
- Do not use this product near water, for example, near a bathtub, washbowl, kitchen sink, or laundry tub, in a wet basement, or near a swimming pool.
- Avoid using a telephone (other than a cordless type) during an electrical storm. There may be a remote risk of electric shock from lightning.
- Do not use this telephone to report a gas leak in the vicinity of the leak.
- Use only the power cord and batteries indicated in this manual. Do not dispose of batteries in a fire. They may explode. Check with local codes for possible special disposal instructions.


Safety Instructions for Batteries

Caution: Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.

- I. Use only two (2) AAA alkaline batteries in your neckloop.
- 2. Do not dispose of the battery in a fire as it may explode. Check with local codes for possible special disposal instructions.
- 3. Do not open or mutilate the battery. Released electrolyte is corrosive and may cause damage to the eyes and skin. It may be toxic if swallowed.
- 4. Exercise care in handling batteries in order not to short the battery with conducting materials such as rings, bracelets and keys. The battery or conduction material may over-heat and cause burns.

Contents and Parts Checklist

Neckloop 3.5mm adapter (2) AAA batteries User Guide **Battery Door** Screw

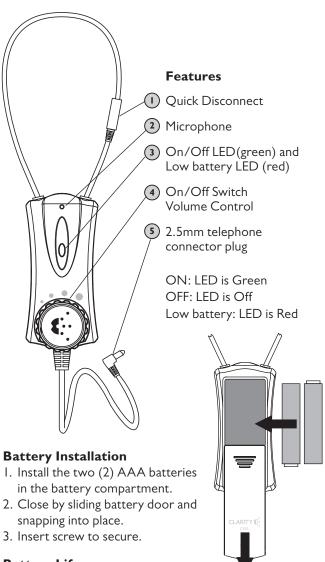

Battery Installation

Battery Life

The battery life will depend upon usage and volume levels. The life of the batteries should last up to 30 to 40 hours before needing replaced.

Volume Control

The CE30 provides up to 30 dB of amplification. The volume is controlled by the power/volume dial located on top of the neckloop.

To Make or Receive a Call

- I. Place neckloop around neck by connecting the quick disconnect. Be sure that the quick disconnect is closed securely in order for the CE30 to work properly.
- 2. Connect the 2.5mm plug into a telephone's 2.5mm headset jack.
- 3. Turn your hearing aid to "MT" or "T" mode.
- 4. While unit is around neck, turn the Power/Volume dial to the left for power to turn ON. The green LED will appear.
- 5. Make or Receive a call using the telephone. (For your telephone's operation, please see its user guide)
- 6. The CE30 features a built-in microphone, allowing for hands-free communication when plugged into a 2.5mm headset jack.
- 7. Adjust the volume to desired level using the dial on top of unit. **Note:** The volume on the telephone may also need to be adjusted.
- 8. When call is complete, turn dial off and unplug the adapter from telephone.
- 9. Please reset the hearing aid if needed.

Important: If your telephone does not have a 2.5mm jack an adapter may be needed. Check with your telephone's manufacturer.

Adapter

The CE30 comes with a 3.5mm adapter. This allows the user to use the neckloop with a 3.5 audio equipped telephone, CD player, MP3 player or any other device that contains an 8-16 Ohm audio output through a 3.5mm jack.

Note: When the CE30 is plugged into a 3.5mm port on a telephone, the microphone will not pick up your conversation. You must speak into the receiver on the telephone.

FCC REGISTRATION INFORMATION

- This equipment complies with Part 68 of the FCC Rules.
 On the back of this equipment is a label that contains, among other information, the FCC registration number and Ringer Equivalence Number (REN) for this equipment. If requested, provide this information to your telephone company.
- 2. The registration jack USOC for the equipment is RIIIC.
- 3. Your telephone company may make changes in its facilities, equipment, operations, or procedures that could affect the operation of your equipment. If they do, you will be given advance notice so as to give you an opportunity to maintain uninterrupted service.
- 4. If you experience trouble with this equipment, please contact Clarity Customer Service Department at I-800-426-3738 for repair/warranty information. If your equipment is causing harm to the telephone network, the telephone company may request that you disconnect the equipment until the problem is resolved.

This equipment complies with Industry Canada CS03.

The Load Number of the equipment is 20. The standard connecting arrangement code for the equipment is CAIIA. **NOTICE:** The Canadian Department of Communications label identifies certified equipment. This certification means that the equipment meets certain telecommunications network protective operational and safety requirements. The Department does not guarantee that the equipment will operate to the user's satisfaction.

Before installing this equipment, users should ensure that it is permissible to be connected to the facilities of the local telecommunications company. The equipment must also be installed using an acceptable method of connection. The customer should be aware that compliance with the above conditions may not prevent degradation of service in some situations.

Repairs to certified equipment should be made by an authorized Canadian maintenance facility designated by the supplier. Any repairs or alterations made by the user to this equipment, or equipment malfunctions, may give the telecommunications company cause to request the user disconnect the equipment. Users should ensure for their own protection that the electrical ground connections of the power utility, telephone lines and internal metallic water pipe system, if present, are connected together. This precaution may be particularly important in rural areas.

Caution: Users should not attempt to make such connections themselves, but should contact the appropriate electric inspection authority, or electrician, as appropriate.

WARRANTY & SERVICE

The following warranty and service information applies only to products purchased and used in the U.S. and Canada. For warranty information In other countries, please contact your local retailer or distributor.

Limited Warranty

Clarity, a division of Plantronics, Inc. ("Clarity") warrants to the original consumer Purchaser that, except for limitations and exclusions set forth below, this product shall be free from defects in materials and workmanship for a period of one (I) year from the date of original purchase ("Warranty Period"). The obligation of Clarity under this warranty shall be limited to repair or replacement, at Clarity's option, without charge, of any part or unit that proves to be defective in material or workmanship during the Warranty Period.

Exclusions from Warranty

This warranty applies only to defects in factory materials and factory workmanship. Any condition caused by accident, abuse, misuse or improper operation, violation of instructions furnished by Clarity, destruction or alteration, improper electrical voltages or currents, or repair or maintenance attempted by anyone other than Clarity or an authorized service center, is not a defect covered by this warranty. Telephone companies manufacture different types of equipment and Clarity does not warrant that its equipment is compatible with the equipment of a particular phone company.

Implied Warranties

Under state law, you may be entitled to the benefit of certain implied warranties. These implied warranties will continue in force only during the warranty period. Some states do allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

Incidental or Consequential Damages

Neither Clarity nor your retail dealer or selling distributors has any responsibility for any incidental or consequential damages including without limitation, commercial loss or profit, or for any incidental expenses, expenses, loss of time, or inconvenience . Some states do not allow exclusion or limitation of incidental or consequential damage, so the above limitation or exclusion may not apply to you.

Other Legal Rights

This warranty gives you specific legal rights and you may also have other rights which vary from state to state.

How to Obtain Warranty Service

To obtain warranty service, please prepay shipment and return the unit to the appropriate facility listed below.

In the United States:

Clarity Service Center 4289 Bonny Oaks Drive, Suite 106 Chattanooga, Tennessee 37406 Tel. (423) 622-7793 or (800) 426-3738 Fax: (423) 622-7646 or

(800) 325-8871

In Canada:

Plantronics Service Center 8112 Trans Canada Highway Ville Saint-Laurent Quebec H4S IM5 Tel: (800) 540-8363 (514) 956-8363 Fax: (514) 956-1825

Please use the original container, or pack the unit(s) in a sturdy carton with sufficient packing material to prevent damage. Include the following information:

- A proof-of-purchase indicating model number and date of purchase.
- 2. Bill-to address
- 3. Ship-to address
- 4. Number and description of units shipped
- 5. Name and telephone number of person to call, should contact be necessary
- 6. Reason for return and description of the problem. Damage occurring during shipment is deemed the responsibility of the carrier, and claims should be made directly with the carrier.

Clarity, a Division of Plantronics, Inc. 4289 Bonny Oaks Drive - Chattanooga, TN 37406 (800) 426-3738 www.clarityproducts.com

©2008 Clarity, a Division of Plantronics, Inc. All Rights Reserved. Clarity, the amplification device, Clarity and the amplification device combined, Amplifying Your Life, and Clarity Power are trademarks or registered trademarks of Plantronics, Inc.

Rev. B (9-08)